

CLEAR MOULDED TRAY



The **Advanced Mobility Systems** moulded trays are currently available in two sizes. The clear tray is thermoformed with a high strength and durable plastic. The result is a strong low profile tray with all smooth edges and rounded corners. Two cup holders on the front left and right side are added for convenience, with a spill edge and channel for containment. Clear spacers and armrest tray rails allow for clearer view and appearance.



WARNING

Check all parts for shipping damage before using. If the product is damaged, do NOT use the equipment. Contact the Dealer for directions and return policy.

DESIGN BENEFITS

- No assembly required
- Light weight
- Deep torso cut out for ease of fit
- Space saving shape and low profile design
- Thermoformed contoured edge for strength

INSTALLATION INSTRUCTIONS

This product comes completely assembled. Simply measure the outside dimension from armrest to armrest on the wheelchair. Adjust the armrest rails on the tray to match the same dimensions. Remember to center the rails by measuring the space on each side of the tray and leave a 1/4" gap to make it easier to slide on and off.

TRAY SIZES

- 24" x 21 1/2" (61 x 56cm) should fit most wheelchairs 15 -18"
- 27" x 21 1/2" (69 x 56cm) should fit most wheelchairs 19 - 22"

WEIGHT

24" x 21 1/2" is 5.0 lbs · 27" x 21 1/2" is 5.5 lbs

WARRANTY

One (1) year limited warranty

621 Justus Drive · Kingston, Ontario · K7M 4H5
800.661.6716 tel · 613.384.7460 fax · 613.634.3990
email · info@amstilt.com www · amstilt.com



ADVANCED MOBILITY SYSTEMS